



## Eclipse 5 Oxygen Concentrator



A : Flow Metre and Flowrate Control

B: Screen

C: Mode Button

D: Power Button (Switch On/Off)

E: Oxygen Outlet Port

### Breathing Accessories:



### How to operate the Oxygen Concentrator:

1. Switch On the Power Button (D)
2. Adjust the Flowrate (A) according to doctor's advise
3. Connect breathing accessories to the Oxygen Outlet Port (E)
4. Let the Oxygen Concentrator run for at least 30 mins. Ensure there are no alarms on screen (B)

<b><u>Do's</u></b> 👍	<b><u>Don'ts</u></b> 👎
<ul style="list-style-type: none"> <li>♥ Keep the Device in a well-ventilated room</li> <li>♥ Keep a 30cm gap between a hindrance and the device filter door</li> <li>♥ Keep the device in an upright position always, even when transporting</li> <li>♥ Change Humidifier water daily</li> <li>♥ Water should be purified water <ul style="list-style-type: none"> <li>o Suitable for drinking water – Boiled/ Distilled/Sterile/RO</li> <li>o If boiled water, ensure water is cooled to lukewarm.</li> </ul> </li> <li>♥ Follow Maintenance &amp; Hygiene Guide</li> </ul>	<ul style="list-style-type: none"> <li>✗ <b>DO NOT USE MORE THAN 2 Hours</b></li> <li>✗ Don't block the airflow of the device. Avoid placing devices against the walls/ furniture/ curtains.</li> <li>✗ Do not share an extension cord with the device and any other appliances.</li> <li>✗ Do not allow kids/pets near the device.</li> <li>✗ Don't keep the device inside a closed compartment</li> <li>✗ Do not use oil or grease on the device</li> </ul>
<p style="text-align: center;">🔥 <b><u>FIRE SAFETY</u></b> 🔥</p> <ul style="list-style-type: none"> <li>✗ <b>DO NOT Smoke near the device</b></li> <li>✗ <b>DO NOT place the device near an open flame/source of heat area</b></li> </ul>	

## Daily

### Humidifier Canister



*Water **MUST** be changed daily*

- ☐ **Clean the Canister:** Rinse with running water, using a sponge and soap to ensure its thoroughly cleaned.
- ☐ **Fill the Canister:** Use one of the following types of water: drinking water, distilled water, sterile water, or reverse osmosis water.

**AVOID: Mineral or Saline Water**

**Replace Cannister every 2 Months**

## Weekly / Monthly

### Device



### Weekly

- ☐ **Clean the Device Exterior:** Wipe down with a soft, damp cloth to remove dust.
- ☐ **Clean the Filter Foam:** Use water or soapy water to clean, then dry thoroughly. Replace the foam if it's faulty.



### Monthly

**HEPA Filter Compartment:** Wipe down with a soft, damp cloth to remove dust.

### Oxygen Cannula / Masks



### Monthly

**Oxygen Cannula or Masks:** Replace monthly, or sooner if they are faulty or very dirty

## Yearly

### HEPA Filter



### Yearly

**HEPA Filter:** Replace yearly.

\*Durability of this product is dependent on the product condition and environment\*

**Note:** The durability of filters and accessories depends on product condition and environmental factors.

### For Service Enquiries:



**Please Call or What's App, +65 6100 3686**