

Daily

Humidifier Canister



Water MUST be changed daily

- ☐ **Clean the Canister:** Rinse with running water, using a sponge and soap to ensure its thoroughly cleaned.
- ☐ **Fill the Canister:** Use one of the following types of water: drinking water, distilled water, sterile water, or reverse osmosis water.

AVOID: Mineral or Saline Water

Replace Cannister every 2 Months

Weekly / Monthly

Device



Weekly

- ☐ **Clean the Device Exterior:** Wipe down with a soft, damp cloth to remove dust.
- ☐ **Clean the Filter Foam:** Use water or soapy water to clean, then dry thoroughly. Replace the foam if it's faulty.



Monthly

HEPA Filter Compartment: Wipe down with a soft, damp cloth to remove dust.

Oxygen Cannula / Masks



Monthly

Oxygen Cannula or Masks: Replace monthly, or sooner if they are faulty or very dirty

Yearly

HEPA Filter



Yearly

HEPA Filter: Replace yearly.

Durability of this product is dependent on the product condition and environment

Note: The durability of filters and accessories depends on product condition and environmental factors.

For Service Enquiries:



Please Call or What's App, +65 6100 3686