

# EverFlo Oxygen Concentrator

## Hygiene & Maintenance Guide

### Daily

#### Humidifier Canister



***Water MUST be changed daily***

- ☐ **Clean the Canister:** Rinse with running water, using a sponge and soap to ensure its thoroughly cleaned.
- ☐ **Fill the Canister:** Use one of the following types of water: drinking water, distilled water, sterile water, or reverse osmosis water.

**AVOID: Mineral or Saline Water**

**Replace Cannister every 2 Months**

### Weekly / Monthly

#### Device



#### Weekly

- ☐ **Clean the Device Exterior:** Wipe down with a soft, damp cloth to remove dust.
- ☐ **Clean the Filter Foam:** Use water or soapy water to clean, then dry thoroughly. Replace the foam if it's faulty.

#### Monthly

**HEPA Filter Compartment:** wipe down with a soft, damp cloth to remove dust.



#### Oxygen Cannula / Masks



#### Monthly

**Oxygen Cannula or Masks:** Replace monthly, or sooner if they are faulty or very dirty

### Yearly

#### HEPA Filter



#### Yearly

**HEPA Filter:** Replace yearly.

*\*Durability of this product is dependent on the product condition and environment\**

**Note:** The durability of filters and accessories depends on product condition and environmental factors.

### For Service Enquiries:



**Please Call or What's App, +65 6100 3686**