

## **Daily**

### **Humidifier Canister**



### ***Water MUST be changed daily***

- ☐ **Clean the Canister:** Rinse with running water, using a sponge and soap to ensure its thoroughly cleaned.
- ☐ **Fill the Canister:** Use one of the following types of water: drinking water, distilled water, sterile water, or reverse osmosis water.

### **AVOID: Mineral or Saline Water**

### **Replace Cannister every 2 Months**

## **Weekly / Monthly**

### **Device**



### **Weekly**

- ☐ **Clean the Device Exterior:** Wipe down with a soft, damp cloth to remove dust.
- ☐ **Clean the Filter Foam:** Use water or soapy water to clean, then dry thoroughly. Replace the foam if it's faulty.

### **Monthly**



**HEPA Filter Compartment:** Wipe down with a soft, damp cloth to remove dust.

### **Oxygen Cannula / Masks**



### **Monthly**

**Oxygen Cannula or Masks:** Replace monthly, or sooner if they are faulty or very dirty

## **Yearly**

### **HEPA Filter**



### **Yearly**

**HEPA Filter:** Replace yearly.

\*Durability of this product is dependent on the product condition and environment\*

## **For Service Enquiries:**



**Please Call or What's App, +65 6100 3686**



## **Intensity 10 Oxygen Concentrator** **Hygiene & Maintenance Guide**

**Note:** The durability of filters and accessories depends on product condition and environmental factors.

**For Service Enquiries:**



**Please Call or What's App, +65 6100 3686**